

Product Category

Single Use Sterile Vaginal Speculum

Warnings


- In accordance with laws and regulations, this product is intended for use only by trained physicians.
- This product is for single use only and must not be reused or used beyond its intended purpose. Reuse or use beyond the intended purpose may compromise the structural integrity of the device, leading to device malfunction, which could result in patient injury, infection, or death. Additionally, it may cause device contamination, patient infection, or cross-infection. Device contamination may lead to patient infection, illness, or death. The company shall not be held liable for any consequences arising from the reuse or use of this product beyond its intended purpose.

⚠️ Cautions

- Before using this device, you should fully understand the technical principles, clinical application scope and related risks of this device.
- This device must not be used for purposes other than its intended use.
- Please read the instructions carefully before use.
- Please note the expiration date before using this product, as it is prohibited to use after the expiration date.
- Do not use if the small package is damaged.
- This product is for single use only. Please dispose of it as medical waste after use.

Product Structure

The single use sterile vaginal speculum features a rotating shaft structure. The basic type of vaginal speculum consists of upper and lower blades with a rack-and-pinion mechanism, while the four-wing type includes upper and lower blades, left and right support wings, and a locking mechanism. The product is provided in a sterile state, sterilized with ethylene oxide, and is for single use only.

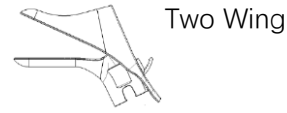

Contraindications

It is contraindicated for those who are allergic to polymer materials; it should be used with caution by unmarried women; and it should be used with caution by other people who are not suitable for vaginal examination.

Specifications and Models

Specifications and models	Blade length L (mm)	View width W (mm)	Blade width W (mm)		View Height H (mm)	
	Size	Size	Size	Tolerance	Size	Tolerance
ST-DVS-S	≥90	≥24	20	±2	35	±5
ST-DVS-M	≥98	≥28	28	±2	45	
ST-DVS-L	≥110	≥30	33	±3	50	
ST-DVS-FW	≥98	≥28	20	±2	50	
ST-DVS-FW2	≥98	≥28	20	±2	50	

How to use

Tear open the small package along the seal, remove the corresponding size of the Single Use Sterile Vaginal Speculum (hereinafter referred to as the Speculum), and have a specialist perform the operation. The specialist will unlock the Speculum, close it, lubricate it, and slowly insert it into the vagina along the posterior vaginal wall. Then, push it upward and inward, while flattening and opening the Speculum. After exposing the examination area, the specialist will fasten the stopper and begin the relevant operation.

Storage and transportation conditions

Store and transport in a cool, dry, well-ventilated and clean environment with relative humidity not exceeding 80% and no corrosive gas.


















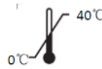

After-sales Service

If you need to inquire about the manufacturing quality, please keep the small packaging bag of the product and the certificate of conformity for contact.

Date: 12 September 2025

Rev. A/1

Symbol Descriptions

	Unique device identifier
	Sterilized using ethylene oxide
	Do not reuse.
	In this document, indicates conditions or practices that could damage the equipment or other property ("CAUTION")
	Do not re-sterilize
	Do not use if package is damaging and consult instructions for use
	Single sterile barrier system
	Medical device
	Use-by date
	Date of manufacture
	Batch code
	On the product, means consult accompanying documentation.
	Catalogue number.
	Keep dry
	This end up
	Fragile
	Keep away from sunlight
	Storage Temperature limits 0°C - 40°C (32°F - 104°F)
	Storage Relative humidity not exceeds 80%

Validity period

Two years. Check the expiry date on the packaging label before use.

Production and Sterilization Date/

Expiration Date

Please refer to the product packaging label.



Legal Manufacturer: Suzhou Sunland Medical Co., Ltd.
Room 403-1, Building 1, No.188 Fuchunjiang Road, Suzhou, China

Entrusted Manufacturer: Suzhou Youkaite Medical Technology Co., Ltd.

West side of the first and second floors, Building B, No. 33 Xinting Road, Suzhou High-tech Zone

 SUNGO Europe B.v.

Fascinatio Boulevard 522, Unit 1.7, 2909VA capelle aan den IJssel, The Netherlands